




Microlux DW Quick Start Guide

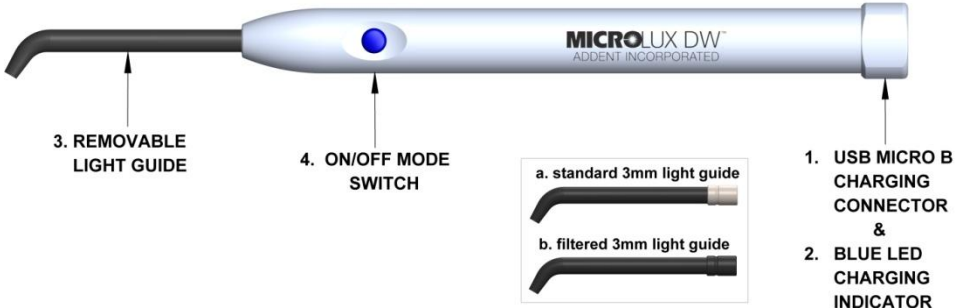

1. Connect the USB charger and fully charge the Microlux DW prior to using the unit.
2. The blue LED Charging Indicator will turn off when the unit is fully charged. Disconnect the charger.
3. Select the appropriate light guide and activate the unit:
 - a. **Visible white light transillumination mode for visualizing cracks, fractures, calculus and carries*** - use the standard 3mm light guide with **SILVER** ferrule and press the ON/OFF switch once.
 - b. **Near UV fluorescence mode for visualizing biofilm, carries bacteria, plaque and non-fluorescent restorative materials*** - use the filtered 3mm light guide with **BLACK** ferrule and press the ON/OFF switch twice.
- In either mode, press and hold the ON/OFF switch for 2 seconds to turn unit off.
- *Refer to User Guide for additional clinical applications.
- Refer to Instructions for Use for additional information and cleaning, disinfection and sterilizing instructions.